For more information  
on this product please  
scan QR code

## Instructions For Use

Návod k použití • Gebrauchsanweisung • Instrucciones de uso  
Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso  
Gebruiksaanwijzing • Instruksjoner for bruk • Instrukcja użycia  
Instruções de uso • Användningsinstruktioner

## Disposable Camera Cover

<b>CZ</b>	Jednorázový obal na kameru
<b>DE</b>	Einweg-Kameraabdeckung
<b>ES</b>	cubierta desechable de la cámara
<b>FR</b>	Housse jetable de caméra
<b>GR</b>	Κάλυμμα κάμερας μίας χρήσης
<b>IT</b>	Copertura monouso per telecamera
<b>NL</b>	Wegwerpbare camerahoes
<b>NO</b>	Engangs kameraoppdekking
<b>PL</b>	Ośłona jednorazowej kamery
<b>PT</b>	Cobertura de câmara descartável
<b>SE</b>	Kameraskydd för engångsbruk

STERILE EO

MD

Single Use

Keep Dry

Keep away  
from sunlight

Do not use  
if packaging  
is opened or  
damaged

### Indications / Intended Use

The Disposable Camera Cover is indicated as a sterile barrier between a sterile telescope and non-sterile camera and lead.

### Instructions For Use

1. Peel open pouch and remove product using standard aseptic techniques.
2. Using the cardboard insert, form an opening for the camera.
3. Insert the non-sterile camera into cavity using standard aseptic technique.
4. Firmly grasp semi-draped camera and pull plastic sleeve over camera lead and continue extending the drape until either:
  - o The lead is covered beyond the sterile field, or
  - o The plastic sleeve is fully extended (recommended).

### Additional Instructions for Use - DCC8007, DCC8008 Only

1. With the drape lever at position 'I' compress the adaptor clip and attach to scope head.
2. Release the adaptor clip and slide the lever to the 'L' position to lock in place.
3. If required the device may be unlocked and repositioned.
4. At the end of the surgery release the lock and compress the adaptor clips to disengage the adaptor from the camera head and remove from sleeve.

### Precautions

Do not use if package is compromised/damaged.

This device may only be used by suitably trained healthcare personnel.

Not to be used for any other purpose.

Single use only, do not re-use, re-process or re-sterilize as this may compromise the performance of the device or create a risk of contamination.

Ensure the device is free from perforations or tears prior to use.

This device is sterile (ETO). Please note, this IFU pertains to Sterile devices sold by Fairmont Medical. If the device has code "XXXX NS", please refer to 3rd party manufacturer for sterility specifications.

This device must be disposed of in accordance with local environmental laws and regulations.

### Materials

The DCC range is made primarily of polyethylene, ABS and polymethyl methacrylate polymer.

All components and packaging are latex free.

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---

Fairmont Medical Products Pty Ltd  
90 Prosperity Way  
Dandenong South  
Victoria 3175  
Australia  
Ph +61 3 9720 8840  
[www.fairmontmedical.com.au](http://www.fairmontmedical.com.au)

Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Fairmont Medical Products Pty Ltd is certified to ISO 13485: 2016

PM029 Version 5.0

Issued by L. Wood 16/11/2023

Product specifications subject to change without notice. Pictures shown may vary from actual product.  
© 2021 Fairmont Medical Products Pty Ltd all rights reserved.

\*Please note the CE 0805 mark applies to sterile versions of the product and the CE mark applies to non-sterile versions.