



For more information
on this product please
scan QR code

Instructions For Use

Návod k použití • Gebrauchsanweisung • Instrucciones de uso
Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso
Gebruiksaanwijzing • Instruksjoner for bruk • Instrukcja użycia
Instruções de uso • Användningsinstruktioner

Disposable Ultrasound Probe Cover

STERILE EO



Single Use



CE
0805



Keep away
from sunlight



Do not use
if packaging
is opened or
damaged



Keep Dry

Caution - Read all instructions prior to use**Indications / Intended Use**

The Disposable Ultrasound Probe Cover (DUC) is a sterile, single-use device that is used to eliminate the risk of transmission of pathogens between device and patient. It is used to cover the clean ultrasound probe during intraoperative / trans vaginal / trans anal procedures.

Instructions for use

1. Peel open sealed pouch using standard aseptic techniques.
2. Take Ultrasound Probe Cover out of inner bag, along with any other accessories such as the blue elastic band(s).
3. Non-sterile nurse holds transducer while sterile nurse covers the transducer with Ultrasound Probe Cover.
4. The transducer should be placed inside ultrasound probe cover by the non-sterile nurse.
5. To prevent wrinkling of the cover, the sterile nurse may flatten surface area against the transducer. Place non-latex elastic blue band(s) around transducer to hold cover in place (if supplied).
6. For further instructions, please follow transducers instructions and guidelines.

Precautions

This device has been designed for single use only. Reuse, reprocessing or resterilisation may compromise the structural integrity, essential material or design characteristics that are critical to the overall performance of the device. This usage may lead to device failure, which may result in injury to the patient. Reuse, reprocessing, resterilisation or repackaging may also create a risk of contamination of the device and / or cause patient infection or cross infection. Contamination of the device may lead to injury. Do not use if package is compromised/damaged.

This device is sterile (ETO). Please note, this IFU pertains to Sterile devices sold by Fairmont Medical. If the device has code "XXXX NS", please refer to 3rd party manufacturer for sterility specifications.

Suitably trained healthcare personnel may only use this device.

This device must be disposed of in accordance with local environmental laws and regulations.

Can be stored between 4°C to 44°C in a clean, dry environment until use.

Do not store in direct sunlight.

Materials

All covers within the DUC range are sterile.

All components and packaging are Latex free.

All components are made from a combination of Vinyl and/or Polyethylene.



Fairmont Medical Products Pty Ltd
90 Prosperity Way Dandenong
South
Victoria 3175
Australia
Ph +61 3 9720 8840
www.fairmontmedical.com.au



Fairmont Medical Products Ltd
Elizabeth House
28 Baddow Road Chelmsford
Ph +44 (0) 1245 206800
www.fairmontmedical.com

Fairmont Medical Products Pty Ltd is certified to ISO13485:2003

PM065 Version 4.0

Issued by L. Wood 14/11/2023