For more information
on this product please
scan QR code

Instructions For Use

**Návod k použití • Gebrauchsanweisung • Instrucciones de uso
Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso
Gebruiksaanwijzing • Instruksjoner for bruk • Instrukcja użycia
Instruções de uso • Användningsinstruktioner**

Disposable Camera Cover

CZ	Jednorázový obal na kameru
DE	Einweg-Kameraabdeckung
ES	cubierta desechable de la cámara
FR	Housse jetable de caméra
GR	Κάλυμμα κάμερας μίας χρήσης
IT	Copertura monouso per telecamera
NL	Wegwerpbare camerahoes
NO	Engangs kameraoppdekking
PL	Osłona jednorazowej kamery
PT	Cobertura de câmara descartável
SE	Kameraskydd för engångsbruk

Indications / Intended Use

The Disposable Camera Cover is indicated as a sterile barrier between a sterile telescope and non-sterile camera and lead.

Instructions For Use

1. Peel open pouch and remove product using standard aseptic techniques.
2. Using the cardboard insert, form an opening for the camera.
3. Insert the non-sterile camera into cavity using standard aseptic technique.
4. Firmly grasp semi-draped camera and pull plastic sleeve over camera lead and continue extending the drape until either:
 - o The lead is covered beyond the sterile field, or
 - o The plastic sleeve is fully extended (recommended).

Additional Instructions for Use - DCC8007 Only

1. With the drape lever at position 'I' compress the adaptor clip and attach to scope head.
2. Release the adaptor clip and slide the lever to the 'L' position to lock in place.
3. If required the device may be unlocked and repositioned.
4. At the end of the surgery release the lock and compress the adaptor clips to disengage the adaptor from the camera head and remove from sleeve.

Precautions

Do not use if package is compromised/damaged.

This device may only be used by suitably trained healthcare personnel.

Not to be used for any other purpose.

Single use only, do not re-use, re-process or re-sterilize as this may compromise the performance of the device or create a risk of contamination.

Ensure the device is free from perforations or tears prior to use.

This device is sterile (ETO). Please note, this IFU pertains to Sterile devices sold by Fairmont Medical. If the device has code "XXXX NS", please refer to 3rd party manufacturer for sterility specifications.

This device must be disposed of in accordance with local environmental laws and regulations.

Materials

The DCC range is made primarily of polyethylene, ABS and polymethyl methacrylate polymer.

All components and packaging are latex free.

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NOTES

STERILE EO

Single sterile barrier system
with protective packaging inside

MD

Is a medical device

Do not resterilise

Made in Australia

Latex free

Keep away from sunlight

Keep dry

CE*
2460

Notified Body

Do not use if packaging is
damaged

**UK
CA**

UK Conformity Assessed

CE*

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Non-sterile

UK REP

Fairmont Medical Products Ltd
Elizabeth House
28 Baddow Road
Chelmsford
Ph +44 (0 1245 206800
www.fairmontmedical.com

EC REP

Emergo Europe
Westervoortsedijk 60
6827 AT, Arnhem The
Netherlands

PM029 Version 4.4
Issued by L. Wood 07/02/2025

 Fairmont Medical Products Pty Ltd
90 Prosperity Way
Dandenong South
Victoria 3175
Australia
Ph +61 3 9720 8840
www.fairmontmedical.com.au

Fairmont Medical Products Pty Ltd is certified to ISO 13485: 2016

Product specifications subject to change without notice. Pictures shown may vary from actual product.

© 2024 Fairmont Medical Products Pty Ltd all rights reserved.

*Please note the CE 2460 mark applies to sterile versions of the product and the CE mark applies to non-sterile versions.